




1. Agitate the tube to coat
tube with anticoagulant

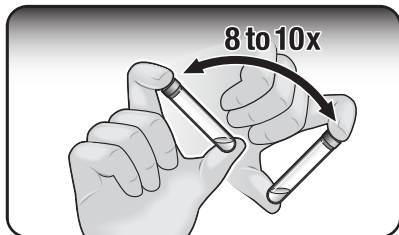


2. Draw blood

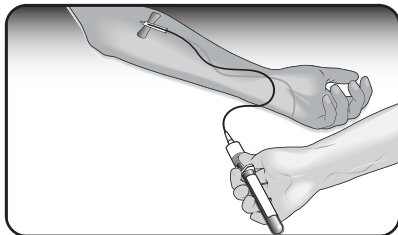


3. Gently shake immediately
after blood draw to mix well

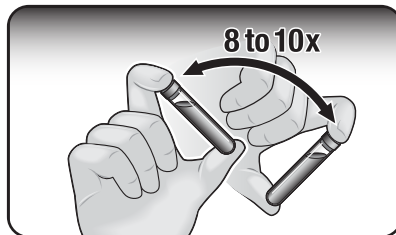


4. Pierce blister

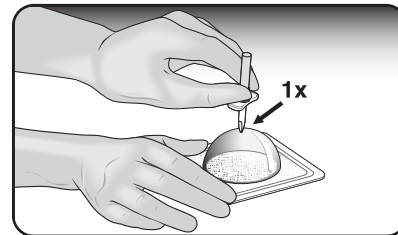


5. Attach safety needle

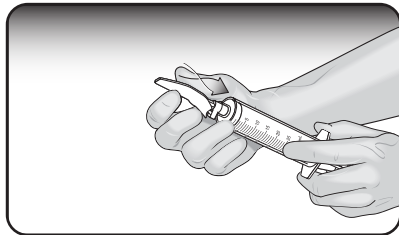


6. Draw up to 10mL of air

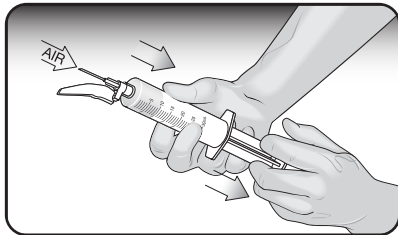


7. Pierce the safety needle into
the cap and withdraw slightly

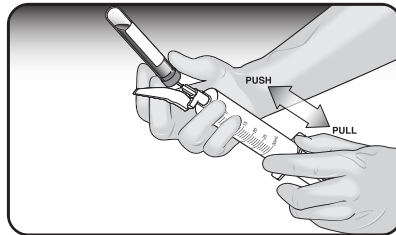


8. Draw blood from ACD-A tube

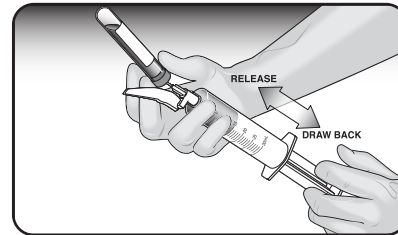


9. Inject blood through the blister hole

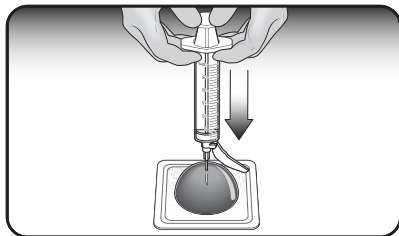

10. Clean and seal the hub

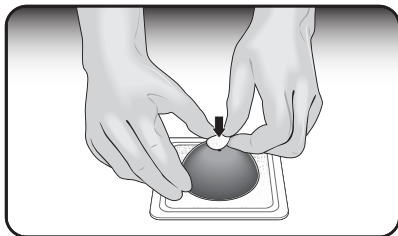


11. Mix contents for 20 seconds

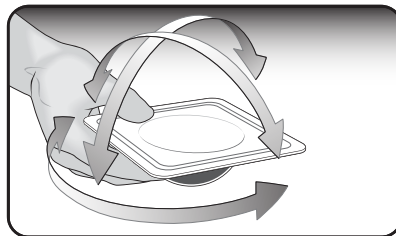


12. Allow at least 5 minutes for coagulation




13. Secure gauze while removing the backing

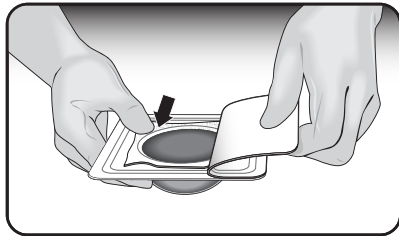


14. Loosen clot tissue with the ring

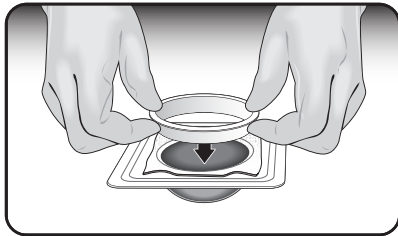

15. Lift clot tissue holding the gauze corners

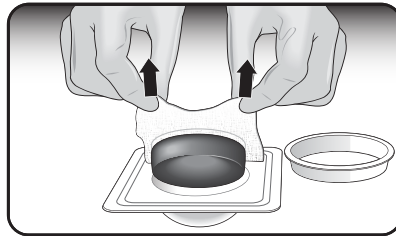

16. Apply clot tissue directly to the wound

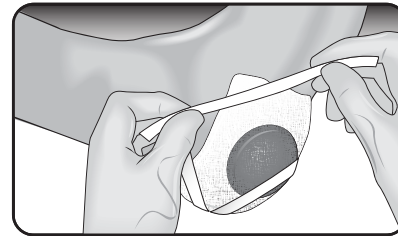

17. Anchor the clot tissue with Medi-Strips, apply non-adherent pad then foam dressing, and secure with gauze | 18. Complete documentation