

# ActiGraft

Autologous Wound Solution

## Patient Discharge Instructions



### PROTECT YOUR DRESSING

#### DO

- Use a cast cover when you shower or bathe to keep the dressing dry.
- **Change the secondary dressing if it becomes saturated with wound drainage.**

#### DO NOT

- Don't get your dressing wet. Avoid placing your wound site under direct flow of water.
- **Don't remove or displace the ActiGraft clot while changing the secondary outer dressing.**

### CARE FOR YOUR WOUND



#### Venous Leg Ulcers

- Elevate the affected leg(s).
- Perform foot and ankle exercises 3x/day minimum.
- Wear compression bandaging.

#### Diabetic Foot Ulcers

- Wear proper offloading shoes on the affected feet.

#### Pressure Ulcers/Injuries

- Avoid direct pressure to the affected area(s) with regular:
  - Repositioning/turning
  - Proper off-loading devices or mattresses.



### CONTACT YOUR PHYSICIAN

#### Contact your physician immediately if the following occur:

- You have increased pain at the wound site.
- You develop a fever.
- Your ActiGraft<sup>+</sup> clot becomes displaced or dislodged.

For safety information, visit [www.legacymedicalconsultants.com/actigraft-safety-info](http://www.legacymedicalconsultants.com/actigraft-safety-info)