

ActiGraft® Procedure Tips

Ensure consistent success and optimal healing outcomes with these practical procedure tips:



Blood Draw Preparation

- Shake the ACDA (blue top) tube before and after drawing blood to evenly distribute the anticoagulant.
- Absolutely NO air in syringe – air can disrupt clot formation.
- Do NOT “bump” air into the ACDA tube – this can cause tube caps to pop and reduce effectiveness.



Clot Formation

- Run your finger around the band aid to ensure adequate seal.
- Shake dome for 20 seconds to initiate clotting cascade.
- Hold mesh firmly when peeling back the Tyvek™ lid to prevent accidental dislodging.
- Use the provided extraction tool to gently separate the formed clot from the dome without tearing.
- Place the clot between two 4x4 gauze pads to absorb excess fluid and allow partial drying before application.



Dressing Application

- Secure with steri-strips, non-adherent dressing, and foam for extra protection.
- Use absorbent secondary dressing to manage exudate and maintain moisture balance.
- Change secondary dressings every 48 hours or as needed to manage the wound environment.

For safety information, visit www.legacymedicalconsultants.com/actigraft-safety-info