


ActiGraft System Treatment Pathway

ActiGraft Treatment Pathway

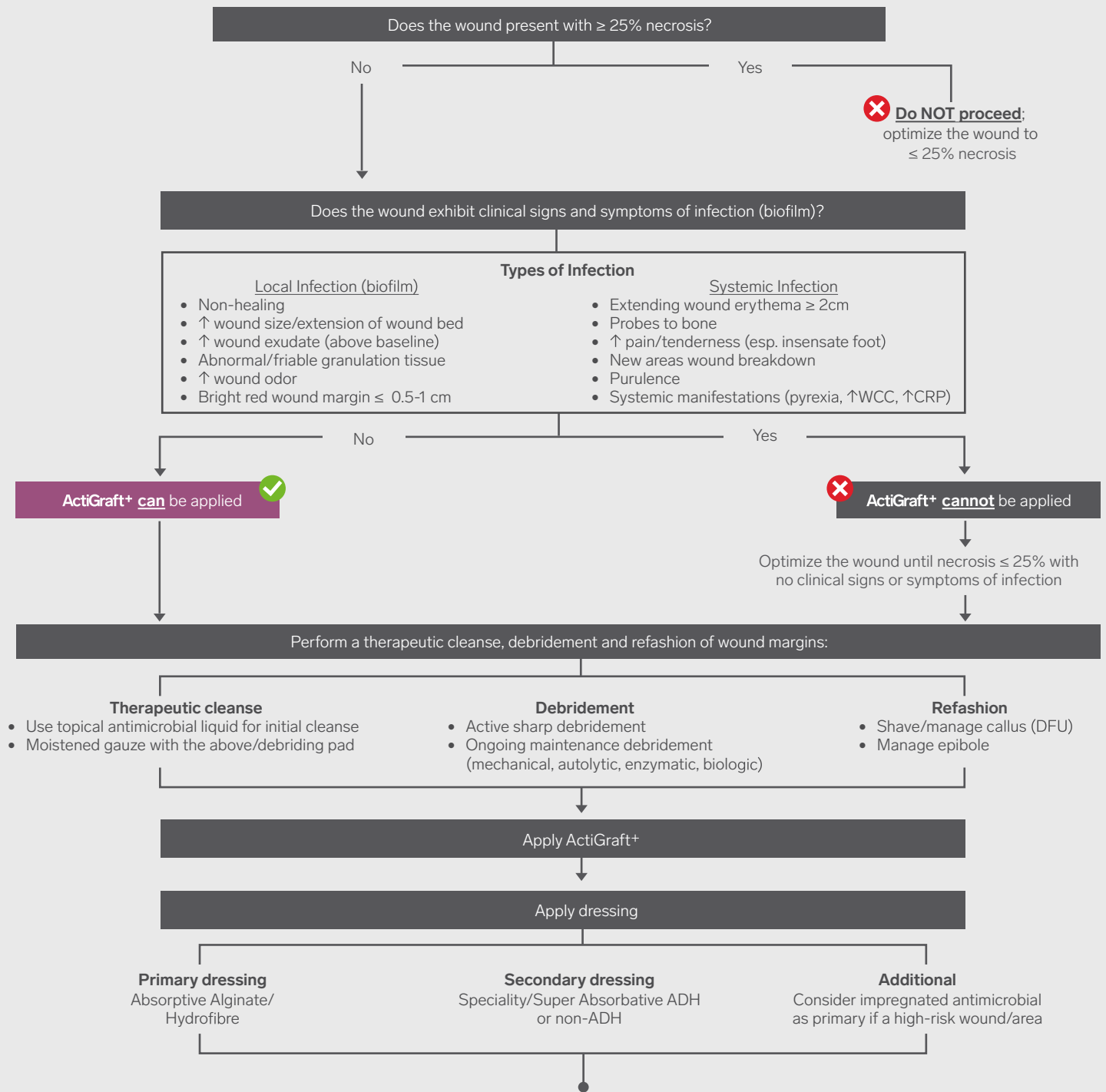

Evaluate the condition of the wound bed and clinical response to ActiGraft+ weekly at each dressing change; if there is wound deterioration or no improvement/reduction in wound volume after 4 applications, discontinue use of ActiGraft+

Considerations	<p>Wound Types</p> <ul style="list-style-type: none"> • VLU – graduated compression 40mmhg (consider speciality absorbent) • DFU – offloading shoe/boot • Arterial/Mixed – confirm sufficient vascular flow to support healing • PU – ensure adequate pressure offloading 	<p>ActiGraft+</p> <ul style="list-style-type: none"> • Change secondary dressings (D2/D3) • ActiGraft+ must remain in place with secondary dressing change • ActiGraft+ is vasoactive to there will be an increase in wound exudate (this is not infective) • Changes in color of wound exudate MUST be investigated

This is not medical guidance. For safety information, visit www.legacymedicalconsultants.com/actigrift-safety-info.

References

- Beitz JM, van Rijswijk L. Using wound care algorithms: a content validation study. *J Wound Ostomy Continence Nurs.* 1999 Sep;26(5):238-9, 241-9. (Literature ID: #US-06-1300)
- Beitz JM, van Rijswijk L. A cross-sectional study to validate wound care algorithms for use by registered nurses. *Ostomy Wound Management* 2010;56(4):46-59.
- Beitz JM, van Rijswijk L. Development and validation of an online interactive, multimedia wound care algorithms program. *J Wound Ostomy Continence Nurs.* 2012 Jan-Feb;39(1):23-34
- Bolton L, McNeess P, van Rijswijk L, de Leon J, Lyder C, Kobza L, Edman K, Scheurich A, Shannon R, Toth M; Wound Outcomes Study Group. Wound-healing outcomes using standardized assessment and care in clinical practice. *J Wound Ostomy Continence Nurs.* 2004 Mar-Apr;31(2):65-71. (Literature ID: #US-07-1013)
- Evidence base for content-validated recommendations underlying Solutions® algorithm. Princeton (NJ): ConvaTec; 2014. 16 p.
- Solutions® algorithms for skin and wound care: customer implementation guide. Princeton (NJ): ConvaTec; 2007 Dec. 4 p. (Literature ID: #US-07-2119)
- Murphy C, Atkin L, Vega de Ceniga M, Weir D, Swanson T, Walker A, Mrozikiewicz-Rakowska B, Ciprandi G, Lázaro-Martínez JL, Černohorská J. Embedding Wound Hygiene into a proactive wound healing strategy. *J Wound Care.* 2022 Apr 1;31(Sup4a):S1-S19. doi: 10.12968/jowc.2022.31.Sup4a.S1. PMID: 35404690.
- Murphy C, Atkin L, Dissemond J, Hurlow J, Tan YK, Apelqvist J, James G, Salles N, Wu J, Tachi M, Wolcott R. Defying hard-to-heal wounds with an early antibiofilm intervention strategy: 'wound hygiene'. *J Wound Care.* 2019 Dec 2;28(12):818-822. doi: 10.12968/jowc.2019.28.12.818. PMID: 31825771.
- Snyder RJ, Bohn G, Hanft J, Harkless L, Kim P, Lavery L, Schultz G, Wolcott R. Wound Biofilm: Current Perspectives and Strategies on Biofilm Disruption and Treatments. *Wounds.* 2017 Jun;29(6):S1-S17. PMID: 28682297.
- Sibbald RG, Elliott JA, Persaud-Jaimangal R, Goodman L, Armstrong DG, Harley C, Coelho S, Xi N, Evans R, Mayer DO, Zhao X, Heil J, Kotru B, Delmore B, LeBlanc K, Ayello EA, Smart H, Tariq G, Alavi A, Somayaji R. Wound Bed Preparation 2021. *Adv Skin Wound Care.* 2021 Apr 1;34(4):183-195. doi: 10.1097/01.ASW.0000733724.87630.d6. PMID: 33739948; PMCID: PMC7982138.
- Gary Sibbald R, Goodman L, Reneeka P. Wound bed preparation 2012. *J Cutan Med Surg.* 2013 Oct;17 Suppl 1:S12-22. doi: 10.2310/7750.2013.wound5. PMID: 24144252.
- Probst S, Apelqvist J, Bjarnsholt T, Lipsky BA, Ousey K, Peters EJG. Antimicrobials and Non-healing Wounds: An Update. *J Wound Management,* 2022;23(3 Sup1):S1-S33. DOI:10.35279/jowm2022.23.03.sup01
- Gottrup, F., Apelqvist, J., Bjarnsholt, T. et al. EWMA Document: Antimicrobials and Non-healing Wounds—Evidence, Controversies and Suggestions. *J Wound Care.* 2013; 22 (5 Suppl.): S1–S92.
- Strohal, R., Apelqvist, J., Dissemond, J. et al. EWMA Document: Debridement. *J Wound Care.* 2013; 22 (Suppl. 1): S1–S52.
- Piaggese A, Bassetto F, den Braber E, Dalla Paola L, Marques A, Palla I, Raspovic K, Scarpa C, Téot L, Triulzi I, Turchetti G. New technologies for tissue replacement; *J Wound Management,* 2023;24(1 Sup1):S1-S130
- European Wound Management Association (EWMA). Position Document: Identifying criteria for wound infection. London: MEP Ltd, 2005
- European Wound Management Association (EWMA). Position Document: Wound Bed Preparation in Practice. London: MEP Ltd, 2004